



MindPulse Test Setup Guide

PATIENT FILE

Before installation, please ensure that you have :

- A **connected 64 bit computer** (windows : Start menu> Settings> System> System settings)
- A good quality **mouse with a cord** (no cordless mice)
- A **different device from the computer used for teleconsultation** with your practitioners : smartphone, tablet.
- A **suitable room** (isolated, no passage, no noise, no distractions).

DURING INSTALLATION

We recommend that you install the test **BEFORE** the consultation to check whether it can be used or not.

1

Here is the MindPulse **download link** :

<https://itsbrain.mindpulse.net/en/telechargement-patients/>

2

We recommend that you install MindPulse with your practitioners's help, **before** the consultation. Installation will create a shortcut launch icon on your screen. By clicking on it, this **message appears** : «*To activate your MindPulse test, you must double-click on the activation file named» activation.mindpulse_config «sent by your clinician as an attachment.».*

MindPulse is correctly installed. Your practitioners has requested your activation file and will email it to you.



Alert and safety messages

Warning messages by your antivirus are simply related to the originality of MindPulse. It is not yet recognized by all operating systems or antivirus programs. As well, your computer may trigger a security alert : **We guarantee that our download platform www.mindpulse.net/ is completely secure.**

On Windows

The simplest case is the appearance of a window stating « *Windows has protected your computer* ». You must click on the link in this window stating « **Further information** », then on « **Run away** ».

On Macintosh

Most often you will get the message «*Unable to open 'Mindpulse' because this application is from an unidentified developer.*»

Click on «**OK**», then go to the system preferences (“cogs” icon),

Open **Security and privacy** (house and wheel icon),

Open the pane «**general**» (Tab).

At the bottom of the window you see «*The opening of 'Mindpulse' is blocked because the application comes from an unidentified developer*», click on the «**Open anyway**» button.

It asks you again, «*Mindpulse is not from an identified developer. Are you sure you want to open it?*» : Click on the «**Open**» button.

For Windows or MAC, you will find on our site :

<https://itsbrain.mindpulse.net/en/mindpulse-download/>

the installation guide concerning problems with the main antivirus.

If you haven't installed MindPulse, tell your practitioner.



DAY OF THE TEST: Supervision by your practitioner

Your practitioner has organized a teleconsultation for you. Before or during the consultation your practitioner will email you the activation file for the MindPulse test.

1

Before consultation :

- Verify there are neither disturbances nor distractions
- Use bathroom facilities beforehand

2

Once you are online with your practitioner, remove any distractions :

Depending on what your clinician will suggest to you, you may

- Put phone on airplane mode
- Close the door for privacy

3

To start your MindPulse test, all you have to do is click on the «activation.mindpulse» file (mindpulse file), which starts your test session.

Execute the test under the supervision of your clinician.

4

At the end of the test :

- Evaluate with your practitioners the quality of the test
- If the result can be retained, click «Save the results»
- Click “Send Results” for send your test data to your practitioner.